## Guide d'utilisation *User's guide*

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*The SMS consists in a technical help you may find needful to use on each overweight patient handling. Under good using patterns, it will prevent care-takers from having work accidents.*

*Before first use, make sure the SMS is carefully cleaned off with clear, soapy water then disinfected with proper hospital detergent-disinfectant.*

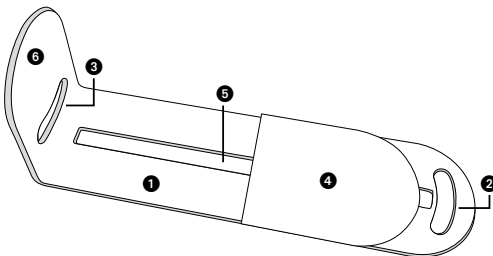
*Dominique and Isabelle (care-taker and nurse) are going to show you throughout the present guide how to use the Slide, as simply as it can be, respectful though of proper gestures and postures protocols.*

*Adapt the height of the bed to your tallness in order to work under good conditions.*

*To optimize the Slide's use, please make sure the device is placed under the right support points of the patient (buttocks and back).*

### MONTAGE ET DÉMONTAGE DU SLIDE MEDICAL SYSTEM™ *ASSEMBLAGE AND DISMANTLE OF SLIDE MEDICAL SYSTEM™*

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*Set the sliding panel into the slit as shown hereabove.*

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*Turn the sliding panel 90° in order that the two extreme located wedges are properly inserted into their slits.*

## MISE EN PLACE SLIDE MEDICAL SYSTEM™ SOUS UN PATIENT ADJUSTING THE SMS™ UNDER A PATIENT

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*Isabelle slightly lifts up the patient's support points to her. Dominique slips the Slide under the first support point keeping the sliding panel at the Slide's extremity, making sure the handle remains free (so that Isabelle can clutch it).*

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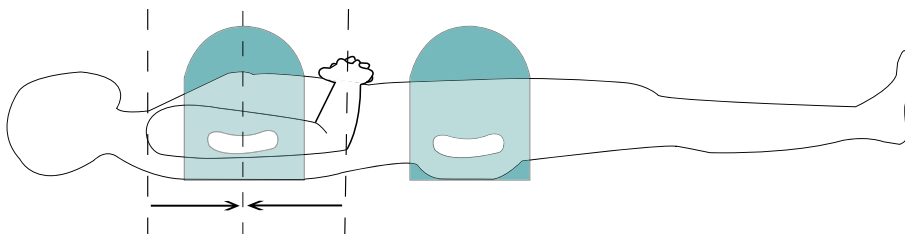
*Dominique slightly lifts up the patient's support point to her, to allow Isabelle to clutch the handle then to pull the Slide to her until it thrust against the patient. Thanks to its sliding design, the Slide prevents friction against patient's skin.*

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*Same method as used previously to place the Slide under the buttocks. In order to help you if there is no medical cons-indication, you may lift patient's knees.*

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*Whatever the application, the Slide should absolutely be placed under the two main support points: THE BACK TOP, between shoulder and elbow, AND THE BUTTOCKS.*

*In no circumstance should the Slide be located above shoulder level, neither be set under a natural body curve.*

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*This technical help allows to place the patient on the side while kept centered in the longitudinal axle of the bed. The patient is correctly prepared to be swung on the side. The lateralization targeted side arm remains free from movement, Slide's thrust is against the patient and the various tools commonly used to block the patient are ready.*

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*Isabelle seizes the two handles and pulls the patient to her, keeping the two Slide panels flat on the support (in order to optimize the sliding effect) and using her own weight swung back, to get the Slide along with her, in total respect with gestures and postures protocols. Isabelle lifts up the two handles to Dominique who seizes them in her turn to help swinging patient's weight.*

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*Since the Slide's shape has a lever effect, it allows to minimize the patient's weight, especially when this latter is overweighted. The own weight of the patient swings him on the side.*

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*Once the lateralization done, withdraw the Slide placing your hands as near to the angle as possible.*

## TRANSFERT À PLAT OU DEMI-ASSIS ET RETRAIT FLAT AND HALF-SEATED TRANSFER AND WITHDRAWAL

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*For a flat or half-seated transfer, one care-taker gently clutches patient's feet while asking this latter to slightly lift up his head (according to the pathology, a third care-taker can block the head and go along with the body while slid over).*

*The second care-taker seizes the handles and transfers the patient effortlessly. Whatever the type of transfer (bed, examination table, stretcher, shower-stretcher, X-rays or scanner table), the principle basically remains the same.*

***It is important to let the Slides laying flat all along its length.***

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*To withdraw a flat positioned Slide from under a patient after a transfer : Isabelle slightly swings the support point to the buttocks level toward Dominique using the flexed leg of the patient (if there is no medical cons-indication). Dominique seizes the withdrawal handle and pulls the Slide to her swinging back her own weight. Operate the same way as for the Slide placed under the patient's back.*

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*Soft brushing of device into disinfecting bath;*

**Laisser agir 15 minutes / Let it take effect 15 minutes**

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*Eliminate overall, single use dishcloth and gloves;*

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➡ Etaler / Spread out dilution;

**Laisser agir 10 minutes / Let it take effect 10 minutes**

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